## ISTRUZIONI D'USO

Italiano

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Fast-curing fluoride varnish

### INDICATION

Prevention of caries, supportive treatment of initial caries, desensitizing treatment of hypersensitive cervical areas and hypersensitive dentine

### ADVANTAGES

- Intensive caries prophylaxis
- Fluoridation of enamel with sodium fluoride and two amine fluorides
- Forms protective film against thermal and mechanical influences

### APPLICATION

1. Surfaces to be treated are thoroughly cleaned and dried with air.
2. Multifluorid must be shaken well before each use. Solid particles must be well dispersed. The extra space provided in the bottle and a glass ball facilitate proper agitation.
3. Multifluorid is applied directly onto a brush or a foam pellet, and then applied thinly onto the area to be treated.

Multifluorid must be so liquid as to run easily from the dropper after shaking. It may be necessary to thin varnish with a few drops of the solvent provided. This solvent is also useful for cleaning instruments etc.

4. The varnish is allowed to be absorbed for 10 - 20 seconds and then dried with air.
5. For hypersensitive necks of teeth and crown edges or similar indications treatment should be repeated two or three times at intervals of 7 days. Surfaces should be dried and touched up again.

Object of every use of Multifluorid must be to keep it as long as possible on the treated surfaces. Please instruct the patient not to brush the treated teeth for 12 hours after application. With correct application and proper dental hygiene Multifluorid varnish remains in place for several days.

### COMPOSITION

Solvent, colloidium, sodium fluoride, Olafur and Dectaflur. Contains rosin.

### STORAGE

Do not use after expiry date.  
Do not store above 20 °C.

### PACKAGING

1 bottle @ 4 g Multifluorid,  
1 bottle @ 10 ml solvent

REF 110140

Federal law (USA) restricts this device to sale by, or on order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only!

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